### Avioq VioOne<sup>™</sup> HIV Profile<sup>™</sup> Supplemental Assay for Confirmation and Differentiation of HIV-1 and HIV-2 Antibodies

C.Chetty, D. Lockwood, T. Holody, KM. Hui, D. Cabral, A. Kozak, J. Jones, K. Popovic, T. Farrar and <u>M. Cronin</u>

Avioq, Inc. March 2019

**Disclaimer: Product FDA and CE Mark Approval Pending** 

#### **HIV Profile Intended Use**


VIOO

- ➤ The VioOne<sup>™</sup> HIV Profile<sup>™</sup> Supplemental Assay (Profile) is an enzyme-linked immunosorbent assay (ELISA) for confirmation and differentiation of individual antibodies directed to various gene products of HIV-1 (Group M & Group O) and HIV-2 in human serum or plasma.
- Profile is intended as an aid in the diagnosis of infection with HIV-1 and/or HIV-2.
- It is intended as an additional, more specific test to confirm the presence of antibodies to HIV-1 and HIV-2 for specimens repeatedly reactive in diagnosis or screening procedures, including pediatric patients (ages 2-20).
- Results of Profile can also be used to distinguish recent from long standing HIV-1 infection and thus be used for HIV-1 incidence estimation.

### **HIV Profile Assay Features**


The VioOne<sup>™</sup> HIV Profile<sup>™</sup> Supplemental Assay contains 3 capabilities

- Detection and confirmation of antibody to HIV-1 antigens
- Detection and confirmation of antibody to HIV-2 antigen

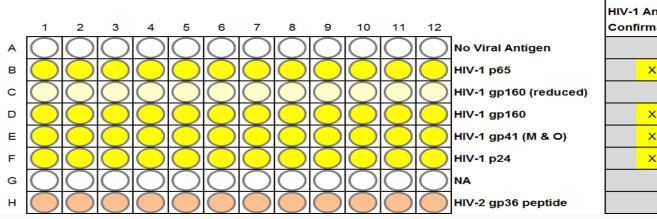

**Aviog** 

-		HIV-2 Antibody Confirmation		Recency Determination				
	х							
							х	
	х						Х	
	х							
	х							
				х				

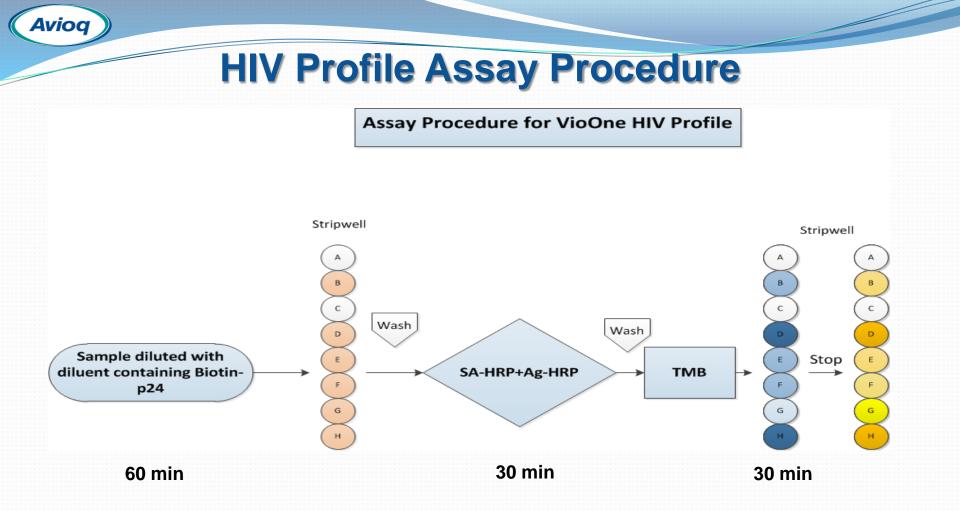

<sup>© 2011</sup> Avioq, Inc. All Rights Reserved

### HIV Profile Analytical Testing (Across 3 Validation Lots)

Preclinical Anticoagulant Study

lvioa

- Serum, EDTA, Citrate, Heparin Plasma from 20 Matched Donors
- Tested Unspiked and Spiked at 3 Levels with HIV-1/HIV-2 Ab
- Repeat Reactive Samples Confirmed Negative (n=100)
- Repeat Reactive Samples Confirmed Indeterminate (n=107)
- Limit of Detection Panels (4 HIV-1 & 4 HIV-2)
- Seroconversion Panels (n=20) & Performance Panels (n=2)
- HIV-1 Group M Clades (n=83) and CRFs (n=13)
- Interfering Conditions (Unspiked & Spiked with HIV-1, HIV-2 and HIV-1/HIV-2, n=14)
- Cross Reactive Conditions / Sample Types (Unspiked Only, n=13)

# **HIV Profile Clinical Testing**

#### Reproducibility

- Seven member panel; Negative (n=2), HIV-1 (n=2), HIV-2 (n=2), HIV-1/HIV-2 (n=1)
- Tested in 9 plates/day; 3 HIV Profile Validation Lots; 1 Analyst
- Duration 5 days at 3 External Sites
- Specificity

Aviog

- 300 Plasma, 300 Serum (Negative in Screening Assays)
- Tested across 3 HIV Profile Validation Lots; 3 External Sites
- Sensitivity
  - 400 HIV-1, 202 HIV-2 (52 at CDC), 50 AIDS, 40 HIV Peds, 10 HIV Coinfection, 10 HIV-1 Group O, 27 HIV Pregnant Females
  - Tested across 4 HIV Profile Validation Lots; 4 External Sites

### **Limit of Detection**

 Analytical sensitivity of Profile was compared to FDA approved predicate Supplemental Assay

**Avioq** 

 Study panel consisted of 4 HIV-1 and 4 HIV-2 positive samples terminally diluted to undetectable antibody levels

				٧	/ioOne™ HIV	Profile™ Assa	iy		
				HIV-1 (n=24)	l.		HIV-2 (n=29)		
			POS	IND	NEG	POS	IND	NEG	
		POS	0	0	0				
ø	HIV-1	IND	2	0	0				
icat	TIV-1	NEG	10	2	10				
Ass		POS				12	0	0	
ē.	HIV-2	IND				5	0	1	
		NEG				1	0	10	

- Profile detected 12/24 HIV-1 samples as positive & 2 as indeterminate
- Predicate detected 0/24 HIV-1 samples as positive & 2 as indeterminate
- Profile detected 18/29 HIV-2 samples as positive & 0 as indeterminate
- Predicate detected 12/292HIX & amples as the siture & 6 as indeterminate

#### **Seroconversion Panels**

 A total of 40 samples from 10 additional commercially available seroconversion panels were tested with Profile and compared to predicate results

	Panel		First Confirmed	Positive Bleed (Days)
Panel ID	Number of Panel Members	Collection Days	VioOne HIV Profile Supplemental Assay	PredicateSupplemental Assay
1	4	17, 38, 49, 51	49	None
2	3	59, 62, 67	67	67
3	3	3, 10, 49	49	49
4	5	4, 9, 18, 21, 25	21	21
5	4	7, 10, 14	None <sup>a</sup>	None <sup>a</sup>
6	5	5, 8, 12, 16, 19	12	12
7	5	10, 14, 18, 21, 25	21	18
8	3	18, 25, 30	25	30
9	4	33, 35, 40, 42	40	40
10	5	0, 4, 7, 25, 31	0	4

<sup>a</sup>Both assays detected panel member 14 as HIV-1 Indeterminate.

**Avioq** 

 Profile detected HIV-1 positive samples within one bleed or better in 10/10 panels for a total of 50.0% (20/40) as reactive compared to Predicate that detected 42.5% (17/40) samples as reactive.

# **Profile Analytical Performance**

#### **HIV-1 Incidence / Prevalence Panel**

 All 15 members of the SeraCare PRB601 Incidence / Prevalence Panel, consisting of 7 known HIV-1 positive incidence (new infections) members and 8 known HIV-1 positive prevalence (long standing infections) members, were found to be HIV-1 antibody positive & HIV-2 antibody negative with Profile

#### HIV-1 / HIV-2 Performance Panel

 All 15 members of the SeraCare HIV-1 / HIV-2 Performance Panel 0800-0331, containing 7 HIV-1 positive, 7 HIV-2 positive, and 1 negative panel member were correctly identified and differentiated by Profile matching Multispot test results

#### **Elevated Level of Biotin**

lvioa

- An HIV negative human serum sample was spiked with 3600 ng/mL of exogenous biotin, which is three times the therapeutic level.
- The human serum sample containing 3600 ng/mL of biotin performed similarly to the same sample without added biotin, regardless whether the samples were spiked or unspiked with HIV antibody (HIV-1 antibody, HIV-2 antibody, or a blend of HIV-1 & HIV-2 antibodies).

# **HIV-1 Group M Clades**

- Nine (9) major HIV-1 Group M clades and 13 CRFs consisting of a total of 96 samples found in the US and Europe were tested positive for HIV-1 with Profile
- 100% positivity rate (95% confidence interval: 96.2% 100%).

Aviog

HIV-1	Number of Samples	Vio	VioOne™ HIV Profile™		
Group M Clade		NEG	IND	POS	
А	10	0	0	10	
В	10	0	0	10	
С	10	0	0	10*	
D	10	0	0	10	
F	11	0	0	11	
G	10	0	0	10	
Н	10	0	0	10	
J	3	0	0	3	
К	9	0	0	9	
CRF_01_AE	4	0	0	4	
CRF02_AG	2	0	0	2	
CRF06_cpx	2	0	0	2	
CRF11_cpx	1	0	0	1	
CRF14_BG	2	0	0	2	
CRF18_cpx	1	0	0	1	
CRF25_cpx	1	0	0	1	
TOTAL	96	0	0	96	

\*1 Clade C sample tested HIV-1 positive with reactivity to HIV-2 antigen

#### Samples Repeatedly Reactive with an FDA Diagnostic or Screening HIV Assay but Confirmed Negative with a Supplemental Assay

- Of 100 Repeatedly Reactive samples confirmed Negative by the Geenius, MultiSpot or IFA, 99 samples were negative and 1 sample was indeterminate with Profile. None of these samples were confirmed positive with Profile.
- All 22 samples confirmed negative by Geenius were confirmed negative with Profile with 100% concordance between the two assays.
- All of 18 samples confirmed negative by MultiSpot were confirmed negative with Profile with 100% concordance between the two assays.
- Of the 60 IFA negative samples, 59 were negative and 1 indeterminate with Profile. Thus, there was 98.33% concordance between the test results of Profile and the IFA assay.

	Supplemental Assay	Supplemental Assay Number		VioOne™ HIV Profile™ Supplemental Assay			
	Results	Number	Negative	Indeterminate	Positive		
Samples Repeatedly	Bio-Rad Geenius <sup>a</sup> NEG	22	22	0	0		
Reactive with a Diagnostic or Screening	MultiSpot <sup>b</sup> NEG	18	18	0	0		
HIV Test	IFA <sup>c</sup> NEG	60	59	1	0		
	Total	100	99 (99.0%)	1 (1.0%)	0		

- <sup>a</sup> Geenius<sup>™</sup> HIV 1/2 Supplemental System
- <sup>b</sup> Multispot HIV-1/HIV-2 Rapid Test

<sup>c</sup> Fluorognost HIV-1 IFA

Avioa

#### Samples Repeatedly Reactive with an Approved Diagnostic or Screening HIV Assay but Indeterminate with a Supplemental Assay

- Of 8 samples confirmed indeterminate by Geenius, 1 was indeterminate, 3 were negative, and 4 were HIV-1 antibody positive by Profile
- Of the 99 samples confirmed indeterminate by IFA, 92 were negative, 3 were indeterminate, 1 was HIV-1 antibody positive and 3 were HIV-2 antibody positive by Profile.
- Also tested were 19 samples which were repeatedly reactive with a diagnostic or screening HIV assay and confirmed positive with an IFA assay, but tested indeterminate with a Western Blot test. All of these 19 samples were confirmed to be positive by Profile.

			VioOne™	HIV Profile <sup>™</sup> Supplementa	I Assay
	Supplemental Assay Result	Number	Negative	Indeterminate	Positive
	Bio-Rad Geenius <sup>a</sup> IND	8	3	1	4
Samples Repeatedly Reactive with a	IFA <sup>b</sup> IND	99	92	3	4
Diagnostic or Screening HIV Test	Total	107	95 (88.8%)	4 (3.7%)	8 (7.5%)
Screening riv rest	IFA <sup>b</sup> POS, Western Blot <sup>c</sup> IND				
		19	0	0	19

<sup>a</sup> Geenius<sup>™</sup> HIV ½ Supplemental System

<sup>b</sup> Fluorognost HIV-1 IFA

Avioa

<sup>c</sup> Cambridge Biotech HIV-1 Western Blot Kit

# **Potential Interfering Factors or Medical Conditions**

- A panel of 130 retrospective samples from patients not known to be infected with HIV representing 13 categories of potentially interfering medical conditions unrelated to HIV infection were tested with Profile.
- Samples were tested unspiked (negative), spiked with an HIV-1 positive antibody, spiked with HIV-2 positive antibody, or spiked with both HIV-1 and HIV-2 positive antibodies.
- Profile results of testing 130 potentially interfering factor samples tested **unspiked** are as follows:

Potentially Interfering Factor	Number	VioO	ne™ HIV Profile	тм
	Tested	NEG	IND	POS
AutoImmune disease	10	10	0	0
Dialysis patients	10	10	0	0
EBV infection	10	10	0	0
HBsAg infection	10	10	0	0
HCV infection	10	10	0	0
High rheumatoid factor	10	10	0	0
Multiparous (pregnant) females	10	9	0	1 <sup>a</sup>
Post influenza vaccine	10	9	1 <sup>b</sup>	0
Yeast (Candida) reactive	10	7	3°	0
Vaccinia vaccine samples	10	10	0	0
HTLV-I/II antibody positive	10	10	0	0
Multiple transfusions	10	10	0	0
Hemophilia	10	0	0	0
TOTAL	130	125 (96.15%)	4 (3.08%)	1 (0.77%)

<sup>a</sup>HIV-2 positive that was negative on duplicate repeat testing.

Aviog

<sup>b</sup>HIV-1 indeterminate that was negative on repeat testing in singlicate due to volume limitation.

<sup>c</sup>Of these three HIV-1 indeterminate samples, one was indeterminate upon repeat testing in duplicate, one was negative upon repeat testing in duplicate and, one was invalid for repeat testing.

#### **Potential Interfering Factors or Medical Conditions**

 Of the 130 potentially interfering factor samples <u>spiked with a blend of</u> <u>HIV-1 and HIV-2 antibodies</u>, all were tested positive for both HIV-1 and HIV-2 antibody with Profile.

**Aviog** 

Potential Interfering Factor	Number		VioOne™ HIV Pro	file™
	Tested	Tested NEG HIV-1 POS		HIV-2 POS
AutoImmune disease	10	0	10	10
Dialysis patients	10	0	10	10
EBV infection	10	0	10	10
HBsAg infection	10	0	10	10
HCV infection	10	0	10	10
High rheumatoid factor	10	0	10	10
Multiparous (pregnant) females	10	0	10	10
Post influenza vaccine	10	0	10	10
Yeast (Candida) infection	10	0	10	10
Vaccinia vaccine samples	10	0	10	10
HTLV-I/II antibody positive	10	0	10	10
Multiple transfusions	10	0	10	10
Hemophilia	10	0	10	10
TOTAL	130	0	130 (100%)	130 (100%)

# **Cross Reactivity Study**

- In a separate cross-reactivity study, a panel of 47 potentially cross-reactive samples representing nine different disease states was tested with Profile.
- All 47 samples were tested negative with Profile, indicating that the samples from these disease states did not result in cross reactivity in the Profile assay.

**Avioq** 

Disease State	Number	VioOn	VioOne™ HIV Profile™		
Samples	Tested	NEG	IND	POS	
Cirrhosis	5	5	0	0	
Hepatitis A	7	7	0	0	
Cancer	5	5	0	0	
HSV IgG	5	5	0	0	
Malaria: P. falciparum	5	5	0	0	
Rubella IgG	5	5	0	0	
Syphilis	5	5	0	0	
Toxoplasmosis IgG	5	5	0	0	
CMV IgG	5	5	0	0	
TOTAL	47	47 (100%)	0	0	

#### Avioq

# **Cross Reactivity Study**

Commercially available samples containing various levels of bilirubin, lipid (triglycerides), hemoglobin, or HAMA (human anti-mouse antibody) were tested with Profile.

 As the table shows, these substances did not cause cross reactivity in the Profile assay.

Panel Member		Number	VioOn	e™ HIV Pro	file™
Samples		Tested	NEG	IND	POS
Total Bilirubin	0.20 mg/dL	1	1	0	0
	2.00 mg/dL	1	1	0	0
	4.00 mg/dL	1	1	0	0
	6.70 mg/dL	1	1	0	0
	11.43 mg/dL	1	1	0	0
Lipemia - Triglycerides	150 mg/dL	1	1	0	0
	272 mg/dL	1	1	0	0
	379 mg/dL	1	1	0	0
	1013 mg/dL	1	1	0	0
	2375 mg/dL	1	1	0	0
Hemoglobin	Normal	1	1	0	0
Ternoglobin	140 mg/dL	1	1	0	0
	275 mg/dL	1	1	0	0
	550 mg/dL	1	1	0	0
	1100 mg/dL	1	1	0	0
НАМА	Negative	1	1	0	0
(Human anti mouse antibody)	Negative	1	1	0	0
	4.0 ng/mL	1	1	0	0
	4.7 ng/mL	1	1	0	0
	7.2 ng/mL	1	1	0	0
	9.6 ng/mL	1	1	0	0
	13.0 ng/mL	1	1	0	0
	27.1 ng/mL	1	0	1*	0
	30.0 ng/mL	1	1	0	0
	38.8 ng/mL	1	1	0	0
	52.7 ng/mL	1	1	0	0
	74.0 ng/mL	1	1	0	0
TOTAL		27	(26/27) 96.30%	(1/27) 3.70%	(0/27) 0.00%

\*HIV-1 Indeterminate that was negative upon repeat testing in duplicate. © 2011 Avioq, Inc. All Rights Reserved

# **Profile Reproducibility Results**

- Of the 1215 replicates tested throughout the precision/reproducibility study (kit controls included), there
  were 7 replicates with test results resulting in incorrect interpretation, i.e., interpretations different from
  expected interpretations.
- Therefore, for Profile, the total percent (%) agreement of observed vs expected results was 99.42% with a 95% CI of 98.82 – 99.77%.
- Predicate showed an overall % agreement of 97.23% with a 95% CI of 96.28 98.00%.

Aviog

	Sample			Tes	t Results	
Code Name	Description	Expected Results	# of Replicates	# of Correct Test Results	% Agreement	95% CI
Kit PC	HIV-1/HIV-2 Antibody Positive	HIV-1 / HIV-2 POS	135	135	100%	97.30% - 100%
Kit NC	HIV-1/HIV-2 Antibody Negative	NEG	135	135	100%	97.30% - 100%
R1	HIV-1/HIV-2 Antibody Negative	NEG	135	134	99.26%	95.94% - 99.98%
R2	HIV-1/HIV-2 Antibody Negative (high background)	NEG	135	132	97.78%	93.64% - 99.54%
R3	HIV-1 Low Antibody Positive	HIV-1 POS	135	133	98.52%	94.75% - 99.82%
R4	HIV-1 Moderate/High Antibody Positive	HIV-1 POS	135	134	99.26%	95.94% - 99.98%
R5	HIV-2 Moderate Antibody Positive	HIV-2 POS	135	135	100%	97.30% - 100%
R6	HIV-2 High Antibody Positive	HIV-2 POS	135	135	100%	97.30% - 100%
R7	HIV-1/HIV-2 Low Antibody Positive	HIV-1 / HIV-2 POS	135	135	100%	97.30% - 100%
	Overall Agreement (including Kit Con	trols)	1215	1208	99.42%	98.82%-99.77%

# **Profile Clinical Specificity**

- A total of 599 serum and plasma, including 20 serum pediatric samples, were collected from individuals at low risk for HIV infection and tested at 3 external sites across 3 Profile validation lots.
- The overall clinical specificity for the VioOne<sup>™</sup> HIV Profile<sup>™</sup> in the low risk population was 98.16% (95% CI: 96.74% - 99.08%).
- Predicate tested a total of 416 serum and plasma samples and showed a clinical specificity of 95.67% and 95% CI of 93.26% - 97.24%.

SAMPLE TYPE	NUMBER	NEGATIVE	INDETERMINATE	POSITIVE
SERUM	279 <sup>A</sup>	272	5 <sup>B</sup>	2 <sup>D</sup>
PEDIATRIC SERUM	20	20	0	0
PLASMA	300	296	3c	1 <sup>E</sup>
TOTAL	599	588 (98.16%)	8 (1.36%)	3 (0.50%)

<sup>a</sup>One sample was repeatedly invalid and excluded from analysis.

lviog

<sup>b</sup>All 5 samples were HIV-1 indeterminate and negative upon repeat testing in duplicate.

°All 3 samples were HIV-1 indeterminate and negative upon repeat testing in duplicate.

<sup>d</sup>Of the 2 samples, 1 was HIV-1 positive and 1 was HIV-2 positive and both were negative upon repeat testing in duplicate. <sup>e</sup>Sample was HIV-2 positive and negative upon repeat testing in duplicate.

# **Profile Clinical Sensitivity Results**

SAMPLE TYPE	NUMBER	POSITIVE	INDETERMINATE	NEGATIVE
HIV-1 POSITIVE	400	398 <sup>A</sup>	1 <sup>B</sup>	1
HIV-2 POSITIVE	202	201 <sup>c</sup>	0	0
HIV-1/HIV-2 COINFECTION	10	10 <sup>D</sup>	0	0
HIV-1 GROUP O	15	15	0	0
AIDS	50	50 <sup>E</sup>	0	0
HIV-1 POSITIVE PEDIATRICS	40	38 <sup>F</sup>	2	0
HIV-1 POSITIVE PREGNANT FEMALES	27	27	0	0
TOTAL	744	739 (99.33%)	3 (0.40%)	1 (0.13%)

<sup>a</sup>One sample was interpreted as HIV-1 positive with reactivity to HIV-2 antigen.

<sup>b</sup>HIV-1 indeterminate and HIV-1 upon repeat testing in duplicate.

**Avioq** 

<sup>E</sup>Eighty-four of 202 samples were interpreted as HIV-2 positive with reactivity to HIV-1 antigens. One (1) HIV-2 sample was interpreted as HIV-1 positive with reactivity to HIV-2 antigen. All other samples were HIV-2 positive.

<sup>d</sup>Ten samples were HIV-2 positive and 6 of 10 were interpreted as HIV-2 positive with reactivity to HIV-1 antigens.

<sup>e</sup>Four samples were interpreted as HIV-1 positive with reactivity to HIV-2 antigen.

<sup>f</sup>One sample was interpreted as HIV-1 positive with reactivity to HIV-2 antigen.

# **Profile Clinical Sensitivity**

vioa

- Serum or plasma (n= 744) repository samples from individuals infected with HIV were tested at 4 external sites and at Avioq with 4 Profile validation kit lots.
- Sensitivity samples consisted of 400 HIV-1 positives, 202 HIV-2 positives, 50 AIDS positive, 40 pediatric HIV positive, 15 HIV Group O positives, 10 HIV-1/HIV-2 coinfected positives and 27 samples from HIV positive pregnant females (7 each from the 1<sup>st</sup> and 2<sup>nd</sup> trimesters and 13 from the 3<sup>rd</sup> trimester).
- The overall clinical sensitivity of the Profile assay against these 744 samples is 99.33% with a 95% CI of 98.44% 99.78%.
- As shown in the Geenius package insert, the overall clinical sensitivity of the Geenius assay against a population of 2024 samples taken from 769 patients is 99.26% with a 95% CI of 98.78% - 99.55%.

# Acknowledgements

#### **Duke University Medical Center**

- Thomas N. Denny
- Heidi Register
- Khalil Itani

**Avioq** 

#### **Clinical Reference Laboratory**

- Mark M. Magee
- Kimbrough Warber

#### University of North Carolina at Chapel Hill, NC

- John L. Schmitz
- Brian Franz

Centers for Disease Prevention and Control

- Timothy C. Granade
- Vickie Sullivan